

## BUTTON FUNCTIONS

### POWER

- ◆ Push **POWER** to turn the processor ON or OFF.
- ◆ Auto off in 8 hours if not used.

### DRYER

- ◆ To restart the dryer, insert a pan, ceph, or 1 to 3 intra-oral film holders into the film processing tank.
- ◆ Close the film processing tank lid. Push **DRYER**.
- ◆ Cancel the dryer by opening the lid.

### DRAIN

- ◆ Push to select one of six drain options by repeatedly pushing **DRAIN**. Push **START** to begin.
1. “**Drain ALL Tanks**” drains all 4 solution tanks, then resets both C: & W: counts on the display to zero.
  2. “**Drain W1 and W2**” drains both water tanks, then resets only the W: count on the display to zero.
  3. “**Drain DEV Only**”
  4. “**Drain FIXER Only**”
  5. “**Drain W1 and the Film Processing Tank**”
  6. “**Drain W2 Only**”

### Developer Temperature

- ◆ Push the up ▲ or down ▼ buttons to adjust the target developer temperature: 72°, 76°, 80°, to 83°.
- ◆ Warmer developer temperatures, faster processing. Cooler developer temperatures, more image detail.

### START

- ◆ Ready to process when the developer temp is between 68° and 85°. ImageMax will select the correct timing.
- ◆ Close the film processing tank lid.
- ◆ Push **START** to begin a processing cycle.

## PROCESSING FILM

1. Open the film processing tank lid.
2. Insert a pan, a ceph, or 1 to 3 film holders.  
\* **First film holder MUST be in the far right position.**
3. Close the lid and press **START**.  
*Okay to open the darkroom door, but keep lights off until pump has turned off (developer has filled the film processing tank).*

## CHANGING WATER

Change the water when the wash water count (W:) reaches between 10-20. When the count is “**W: 21**”, ImageMax will require both water tanks to be drained.

**Tip:** Change the water daily. Doing so extends chemistry life and produces better film quality.

### PROCEDURE:

1. Push **DRAIN** 2 times. On screen: “**Drain W1 and W2**”.
2. Push **START**. Wait for ImageMax to finish draining.
3. Add 3-6 drops of **WaterFlo Plus** into the water tanks.
4. Fill water tanks to the cap opening brim with water.
5. Press **START** to test the system.

## CHANGING CHEMISTRY

Recommend chemistry change at approximately “C: 300” or chemistry age 7-14 days (max 4 weeks).

### PROCEDURE:

1. Push **DRAIN** once. On screen: “**Drain ALL Tanks**”.
2. Push **START**. Wait for ImageMax to finish draining.
3. RINSE: Fill each container about 1/2 way with warm water, gently agitate the bottom of the tanks with the cleaning tool, then drain “**Drain ALL Tanks**” again.
4. Fill each tank to the cap opening brim with the appropriate solution. Stop filling just before it runs out.  
**NOTE:** Add 3-6 drops of **WaterFlo Plus** to both water tanks before adding water to thoroughly mix.
5. Press **START** to test the system.